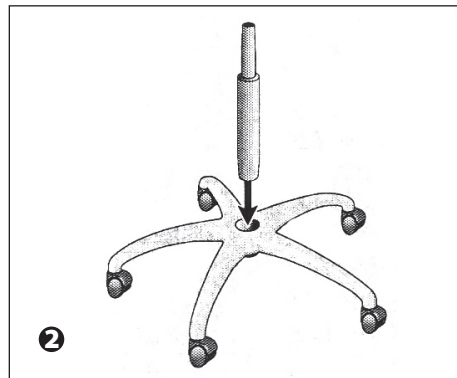
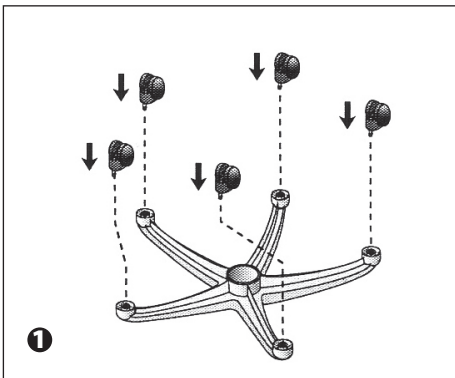
- de** Gebrauchsanweisung – Drehhocker Classic
- en** User's Manual – Swivel Stool Classic
- fr** Mode d'emploi – Tabouret pivotant Classic
- it** Istruzioni per l'uso – Sgabello girevole Classic
- es** Instrucciones de empleo – Taburete giratorio Classic
- pt** Manual de operação – Banqueta giratória Classic
- ru** Руководство по применению – Табурет газлифт «Классик»

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en – Attaching the cap: Apply the cap A precisely. Push down the cap until the rings B and C lie close to each other. If the force to put on the cap is not sufficient, rub the edge below the ring B with a rinsing agent or a lubricant.

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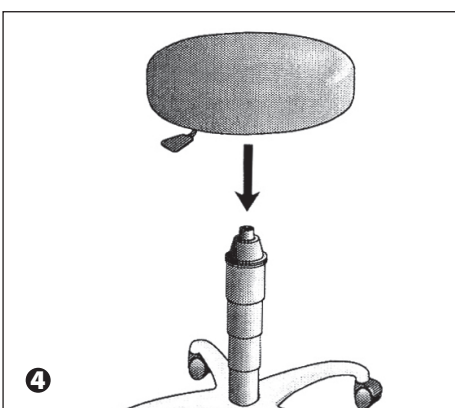
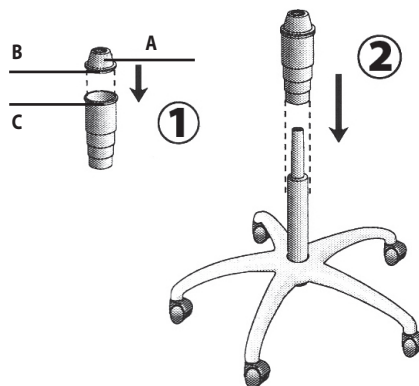
en User's Manual

Purpose: The high-quality swivel stool is designed for use in medical practices and in medical facilities.

Guarantee: When used properly and in accordance with the User's Manual we guarantee this product for two years from the date of purchase. For further information or repairs, please contact your dealer.

! Safety instructions!

- The swivel stool is only to be used when in good working order and condition.
- Please do not stand on the stool – risk of falling!

Technical data:

- Seat height: 450 – 570 mm
- Thickness of cushion: 50 mm
- Overall weight: approx. 7.15 kg
- Carrying capacity: 110 kg max.
- Safety pneumatic spring with lever release
- More information about KaWe products can be found at www.kawemed.de

Cleaning:

- The swivel stool is resistant to disinfectants, urine and blood, and is also abrasion-proof.
- The swivel stool is to be wiped off with a damp cloth and with a surface disinfectant.

Maintenance: When used and cleaned appropriately, the swivel stool remains maintenance-free.

Storage: Store the stool in a dry place.

Subject to technical modifications without prior notice.

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	Manufacturer		Lot code
	Date of manufacture		Caution!

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