
- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

Kopfleuchte HiLight® LED H-800

CE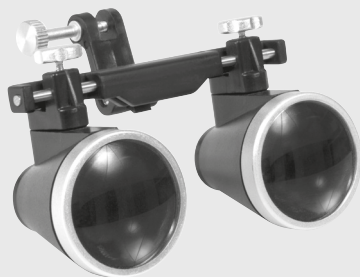

de	Kopfleuchte HiLight® LED H-800	4
en	Headlamp HiLight® LED H-800	6
fr	Éclairage frontal HiLight® LED H-800	8
it	Caschetto HiLight® LED H-800	10
es	Lámpara frontal HiLight® LED H-800	12
pt	Lanterna frontal de HiLight® LED H-800	14
ru	Светодиодный налобный осветитель HiLight® LED H-800	16

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User's Manual Headlamp HiLight® LED H-800

Dear customer, thank you for choosing a KaWe product. Our products are characterised by their high quality and longevity. This KaWe product complies with the regulations prescribed by EG-Guideline 93/42/ EWG (Guideline for Medical Products).

Before using this product, please read this User's Manual carefully and in its entirety and heed the specified maintenance instructions.

Before using this product, make sure to thoroughly familiarise yourself with its operation.

Use: This HiLight LED H-800 headlamp is only to be used by authorised, qualified personnel.

Purpose: The HiLight LED H-800 headlamp is intended to be used as a source of illumination during medical examinations. The headlamp is designed to be worn on the head while being used, thereby providing the wearer with an optimally illuminated visual field. If desired, the binoculars can be attached to the headlamp in order to magnify the examined subject. The following options are available (magnification x working distance): 2.5 x 340mm, 2.5 x 420mm, 3.5 x 340mm, 3.5 x 420mm.

Assembly/Disassembly: The HiLight LED H-800 headlamp must be used with a KaWe-brand lithium-ion rechargeable battery pack. For this purpose, there is a cord on the right-hand side, which is directly connected to the rechargeable battery on the headband as well as with the cord for the rechargeable battery on the belt. When using the headlamp with the rechargeable battery on the headband, the battery pack must first be secured to the back side of the head strap. Insert the latches on the battery pack housing into the slots on the back side of the headlamp strap and gently slide them into their locked position. To unlock and remove, slide in the battery pack in the opposite direction.

The binoculars can be mounted on the front of the headlamp underneath the lamp body, whereby the fastening screw on the binocular hinge must first be removed. After the binoculars have been inserted into the mounting bracket, the fastening screw must be screwed from the front into its threaded hole. The position of the binoculars can be adjusted by gently loosening the fastening screw. After adjustments have been made, tighten the fastening screw to ensure that the binoculars remain in the desired position.

Unintended Use/Contraindication of the Products: Any usage that is different from or that goes beyond the stated intended purpose is considered improper. The manufacturer shall not be liable for damage resulting from improper use; the user alone bears the risk. Only the included rechargeable battery pack or an original KaWe rechargeable battery pack is to be used to supply power to the lamp.

Initial Use/Handling: **1.** Connect the headlamp to a rechargeable battery pack (for headband or belt). **2.** Press and hold the control knob for approx. 1 second. The lamp will illuminate. **3.** After the light has been turned on, the brightness is automatically at its lowest level. In order to increase the brightness, turn the control knob clockwise. LED indicator lights display the selected brightness level. **4.** To turn the light off, press and hold the control knob for approx. 1 second. **5.** The headlamp cannot be used while the batteries are being charged.

Technical Data:

Admissible ambient temperature	+10 °C to +40 °C
Relative humidity	10 to 75%
Storage temperature	+5 °C to +60 °C
Air pressure	700 to 1060 hPa
Protection type	IP X0
Input voltage	100-240VAC, 50/60Hz
Output voltage	8.4 V DC
Output current	max. 800 mA
Weight (without battery)	approx. 160 g

Warning: This device is maintenance free. For this reason, the housing cannot be opened. Repairs are only to be carried out by a qualified expert. Remove and replace damaged parts.

If the device appears to be damaged, disconnect the power cord from the battery pack immediately. Avoid further operation and return the device to our customer service / technical department. The HiLight LED H-800 headlamp is not designed for use in environments in which explosions may occur, in the presence of strong magnetic fields, or in surgery. Only KaWe-brand components are to be used. Modifications to this device are not allowed.

Instructions

Storage and Transport: Please store this device in its packaging prior to assembly in order to avoid contact with impurities or dust.

Cleaning Preparation: No specific requirements.

Manual Cleaning: The external parts of the headlamp and the battery packs can be cleaned with a damp, soft, lint-free cloth. Soapy water or other non-aggressive agents may be used. The lenses on the binoculars can be carefully wiped clean with a glass cleaner.

Automatic Cleaning: No automated cleaning possible/necessary.

Disinfection: The housing of the unit may be wiped with a surface disinfectant. Always wipe the surface dry.

Inspection and Verification of Proper Operation: Inspect the movable parts (such as the joints). Turn the device on. All devices: perform a visual inspection for damage and wear.

Packaging: Packaged in cardboard packaging with shaped cut-outs, packed in cardboard boxes.

Sterilisation: No sterilisation possible/necessary.

Additional Information: Heed the information in the User's Manual. For answers to further questions, contact the manufacturer before using this device.

Manufacturer: **KaWe**

Contacting the manufacturer: Address or telephone number of your dealer, or dial +49-7141-68188-0.

Further Instructions, Maintenance, Storage: When used and stored properly, this product will perform reliably for many years.

Guarantee: When used properly and in accordance with the User's Manual, this product is guaranteed for two years from the date of purchase. For answers to further questions or for necessary repairs, please contact your dealer.

Common Accessories: Further information about accessories can be found on our homepage at: **www.kawemed.de**.

Symbol key:

	Manufacturer
	Date of manufacture
	Heed the User's Manual
	Product reference number
	Separate disposal of electric and electronic devices
	Dispose of expired batteries separately at an appropriate collection centre
	Complies with relevant EU guidelines
	Only use device in dry rooms
	Device safety class: II
	Keep dry
	Electromagnetic field warning
	Type B

Electromagnetic compatibility of product: please refer to www.kawemed.de for further information

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KIRCHNER & WILHELM GmbH + Co. KG