- de** Gebrauchsanweisung
- en** User's Manual
- fr** Mode d'emploi
- it** Istruzioni per l'uso
- es** Instrucciones de empleo
- pt** Manual de operação
- ru** Руководство по применению

Zecken-Fix

CE

QM-1-058M

de	Gebrauchsanweisung „Zecken-Fix“ Zeckenpinzette	4
en	User's Manual „Zecken-Fix“ Tick Tweezers	9
fr	Mode d'emploi « Zecken-Fix » Pince à tiques	14
it	Istruzioni per l'uso „Zecken-Fix“ Pinza per la rimozione di zecche	19
es	Instrucciones de empleo „Zecken-Fix“ Pinzas para garrapatas	24
pt	Manual de operação „Zecken-Fix“ Pinça para carraças	29
ru	Руководство по применению «Zecken-Fix» пинцет для удаления клеща	34

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User's Manual "Zecken-Fix" Tick Tweezers

Dear Customers, thank you for choosing a KaWe product. Our products are known for their high quality and long life. This KaWe product meets EC Standards 93/42/EEC (standards for medical products).

Please read this User's Manual thoroughly and carefully before attempting to use this product and heed the given care instructions.

Use: „Zecken-Fix“ tick tweezers are only to be used in accordance with the instructions given in the User's Manual. Before use, the instructions for preparing the tick tweezers are to be followed. Immediately after the tick has been removed, the bitten part of the body is to be disinfected and in all cases, as a precautionary measure, examined by a doctor. The removal of the tick does not protect against pathogenic germs.

Purpose: "Zecken-Fix" tick tweezers are suitable for use on people and animals and serve to quickly remove ticks. The latin name for ticks is *Ixodes ricinus*. Their habitat is mainly in mildly forested areas with pronounced underbrush. They also live on the edge of forests, mostly on the underside of grass and bush leaves, and rarely higher than 1.5 m above the ground. They live on animal and/or human blood and can go without food for up to 2 years. Their beak buries itself mostly in areas where the skin is moist (scalp, ear, armpit, crook of the arm, between the toes...). This takes place most of the time unbeknownst to the subject as their saliva works as an anaesthetic.

According to medical findings, it is incorrect to try to smother the tick with oil or glue. This method raises the risk of infection as the smothering ticks immediately salivate poisonous pathogenic germs when they lack air.

Therefore: if a tick is discovered on the body, it should be promptly removed with the „Zecken-Fix“ tick tweezers!

The „Zecken-Fix“ tick tweezers have the advantage of having adjustable tweezer tips that can be set by the user to the desired position (see below).

The sooner the tick is removed, the smaller the danger of the transmission of bacteria or viruses becomes.

Fig. 1

Hold the "Zecken-Fix" tick tweezers between the thumb and forefinger. The grabber is opened by pulling back on the sleeve.

Fig. 2

Now slowly pull the tick out. If the tick slips out of the grabber during this process, the process from step 1-2 must be repeated.

Unsuited use/contraindication of the products: Any use other than that described here is not in accordance with the intended use of this product. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

Further information, maintenance, storage: When used and stored properly, the tick tweezers can serve you many years.

Guarantee: When used properly and in accordance with the User's Manual we guarantee this product for two years from the date of purchase. Should you need further information or should your instrument require repair, please contact your dealer.

Common accessory parts: Further information about this article can be obtained on our website: www.kawemed.de.

Manufacturer: **KaWe**

Warning instructions: Prior to use, sterilize the tick tweezers with alcohol or clean them with a disinfectant, in order to eliminate the risk of a – relatively improbable – infection from the tick tweezers. Ticks should only be removed by adults.

If any of the tick's mouthpiece remains in the skin after the tick has been removed, please consult your doctor.

Medical information: Tick bites can cause dangerous medical complications. Therefore, if you are bitten by a tick, consult your doctor. In areas known to have a high risk of tick infection, it is highly recommended that you obtain a vaccination against early summer meningoencephalitis (TBE). If you have a tick bite that results in a migrating red spot around the bite that continues to increase in size, please contact your doctor immediately.

After the tick has been removed, the affected area should always be disinfected. This can be done with alcohol-based disinfectants, such as Sterillium®.

Limitation of reuse preparation: No special requirements

Instructions

Workstation: Remove dirt accumulation on outer surface with a disposable cloth.

Storage and transport: In order to avoid dirt accumulation or dust, always store the tick tweezers in their packaging.

Cleaning preparation: No special requirements.

Manual cleaning: Surface disinfectants and/or cleaning agents, i.e. with pure alcohol or helipur.

Automatic cleaning: Automatic cleaning is not possible.

Disinfection: Surface disinfection is possible with pure alcohol.

Maintenance: No maintenance required.

Check and performance test: The movable parts such as gripping pliers should be checked to ensure easy moveability.

Sterilization: Sterilization in the autoclave up to 134° C is possible (recommended holding time 20 min).

Storage: No special requirements.

Additional information: No special requirements.

Disposal: Dispose along with normal domestic waste.

Contacting the manufacturer: Address or phone number of the dealer or dial the number +49-7141-68188-0.


Symbol key:

	Manufacturer		Heed the User's Manual
	Date of manufacture		Complies with relevant EU-guidelines
	Lot code		Caution!


Adjusting the tweezer tips: Tick tweezers are tools used to facilitate the removal of ticks from the skin. The tips of the Kirchner & Wilhelm tick tweezers are easy to adjust, enabling each user to adjust the instrument to best fit their own technique for removing the tick. These adjustable tips are especially useful when the tweezers must be used at a very low angle relative to the skin's surface

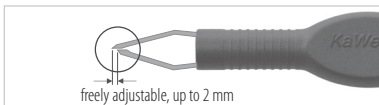1.

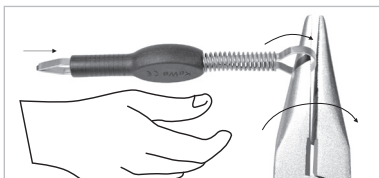2.

3.

short tip
for gripping

Before the tick is removed, one of the tweezer blades can be slid underneath the tick. For this purpose, it can be helpful if the lower blade is slightly longer than the top blade. If, however, the ticks needs only to be grasped by the tweezers, no alignment adjustment is necessary. Please note that the Kirchner & Wilhelm tick tweezers are designed such that the blade tips are freely adjustable. The length of these tips relative to one another can be adjusted using a standard pair of household pliers.

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KIRCHNER & WILHELM GmbH + Co. KG